



## WaterSolver Installation Guide

1. Remove water bottle from dental chair.
2. Cut 3" from water bottle tube.
3. Attach quick connector to WaterSolver and water bottle tube.
4. Fill water bottle with water.
5. Return water bottle tube with WaterSolver to chair.

### NOTES

- Opening of water bottle may require widening to allow for WaterSolver.
- WaterSolver must be fully submerged in water for proper operation. If not, water compressed air and water will be discharged.
- Attach the enclosed label to a water bottle and record the installation date. Reorder every six months by calling (942) 449-3946 or e-mail [scott.ruprecht@gocleanandclear.com](mailto:scott.ruprecht@gocleanandclear.com).

**Clean+Clear Corporation**  
4707 Highway 61, Suite 184 • White Bear Lake, MN 55110  
Phone (952) 449-3946 • Fax (651) 490-3450  
[info@gocleanandclear.com](mailto:info@gocleanandclear.com)